


# Elite Electric Breast Pump Rental Instructions





## Single Pumping

### Collection Kit




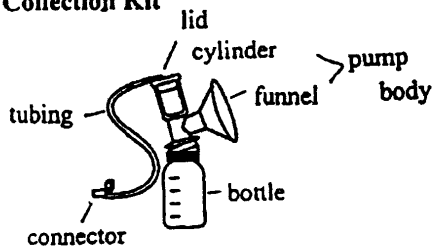


### Assembly of the Equipment

1. Plug your electric pump into an electrical outlet close to your chair.
2. Remove the collection kit from the bag.  
\*If you are converting a One Hand Breast Pump please remove handle assembly. For detailed instructions refer to One Hand Breast Pump Instruction Sheet.
3. Press the connector firmly into the opening found on the electric pump.
4. Attach the tube to one opening of the connector. Make sure the plug is in the other opening of the connector.
5. Push the white valve onto the part found inside the pump body. Handle the valve by holding the thick sides. This valve is very fragile and should be handled with care. Make sure it is on firmly. Screw on the bottle.
6. Put the silicone diaphragm into the cylinder of the pump body.
7. Snap the white cap with tubing attached onto the cylinder of the pump body.
8. Check the connector to make sure it is still firmly on the electric pump.
9. You are now ready to use the pump.

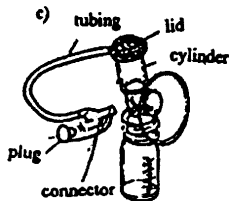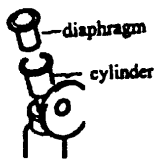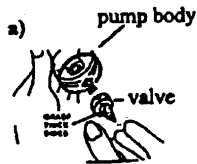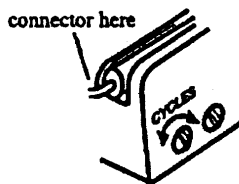

# How to clean your collection kit and Elite electric breast pump

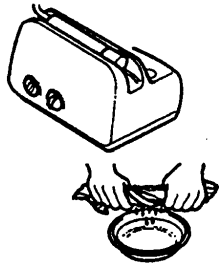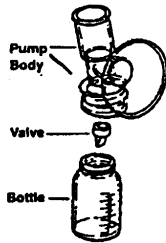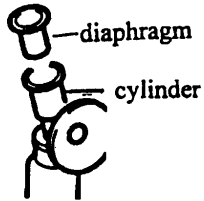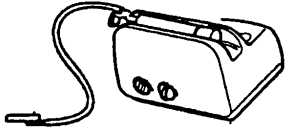

## *Collection Kit:*

1. **Do not clean the tubing.** Leave the tubing on the pump.
2. Take the lid off the cylinder.
3. Remove the diaphragm from the cylinder or lid.
4. Unscrew the bottle from the pump body.
5. Gently remove the valve by holding the thick sides. This valve is very fragile and should be handled with care.
6. Follow instruction pamphlet, inserted with the collection kit, on General Cleaning Instructions and sanitization for the above disassembled parts.

## *Electric Breast Pump:*

1. Wipe over the case only with a damp cloth of soap and water. Remember do not get tubing wet.


## Having Problems?

If your pump stops suddenly or will not start, check to make sure it is plugged in.

If there is little or no suction:

1. Check that the collection kit is put together correctly. Review steps in section titled ***Assembly of the Equipment***.
2. The tubing should not be twisted or kinked
3. Check that the white valve is not torn. This valve is very fragile and should be handled with care. If the valve is ripped or torn you will not feel any suction.

## Double Pumping

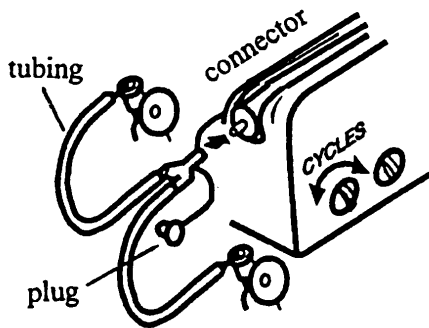



*Please note:*

*You will need two collection kits or a double collection kit to double pump*

Follow the instructions for single pumping, number 1-3.

- 4a. Attach the tubes to the connector (one on each opening).  
The plug will not be used when double pumping.
- 4b. Continue to follow the instructions # 5 under single pumping section, completing the steps with each collection kit.

## Getting Started

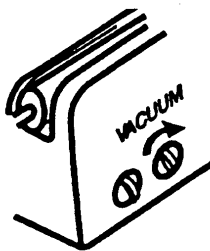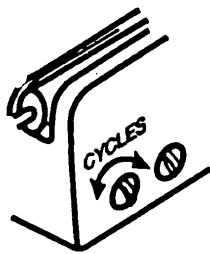

1. The cycle knob should be set at MED setting to start. By turning the knob to the right it increases the speed and turning it to the left decreases the speed.
2. To start the pump, turn the vacuum knob to the ON position. It is recommended that you use the pump on low vacuum (suction) for the first few times. Then slowly turn it until you reach a level that is comfortable for you. The suction increases as you turn the knob to the right.
3. Refer to **Breastfeeding Answers** “How do I use a breast pump” and “How do I pump and store breast milk?” included in your package.